

CHECK CABLE CONNECTIONS



iPad



Power supply



Elisa (slot 1-4)




LAN connection




GE Monitor (slot 1-4)

START SANDMAN.IC

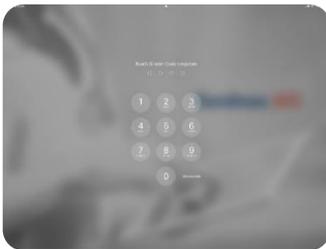

Enter code on iPad

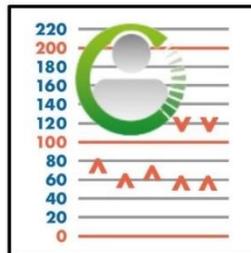

Start Sandman.ICU app



Enter user and password




This project has received funding from the European Union's Horizon 2020 research and innovation programme under grant agreement No 101015930.

LOAD TREATMENT



Enter case number and press  (if not yet loaded)

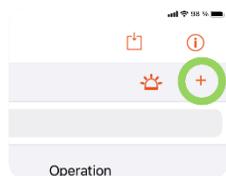

Press  for new treatment




Press blue bar



Enter operation area and start emergency surgery

ESTABLISH DEVICE CONNECTION

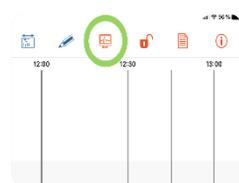


Press 



Select devices and press „connect“




Press „Done“ when entries are green

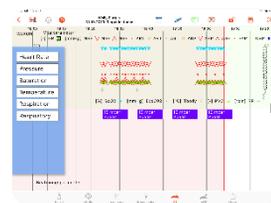

Data is recorded

DISCONNECT DEVICE

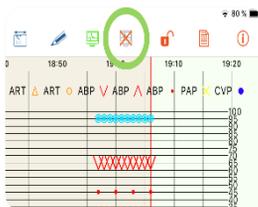


Press 



Confirm warning


SAVE TREATMENT

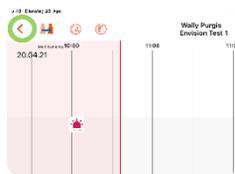

Go to treatment list by pressing 



Press 

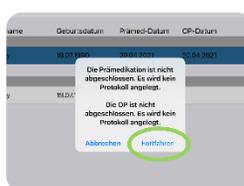

Confirm warning



Wait until treatment appears in „available“ list